

# ICARE BED MOVER TROLLEY

Assembly Instructions

Unpack the components from the carton.

Screw the caster wheels onto the base section, set down on flat surface, and lock all casters.



Fix the Connector Tube to the wheel base section.



Insert the Clamp Bar into the Connector Tube.

Screw the adjuster knobs into the connectors.

Have the adjuster knobs facing the back of the trolley for easy adjustments.

Fit the adjustable clamps onto the clamp bar.

Have the clamp bracket and adjuster knobs facing away from where the bed will sit to prevent damage.

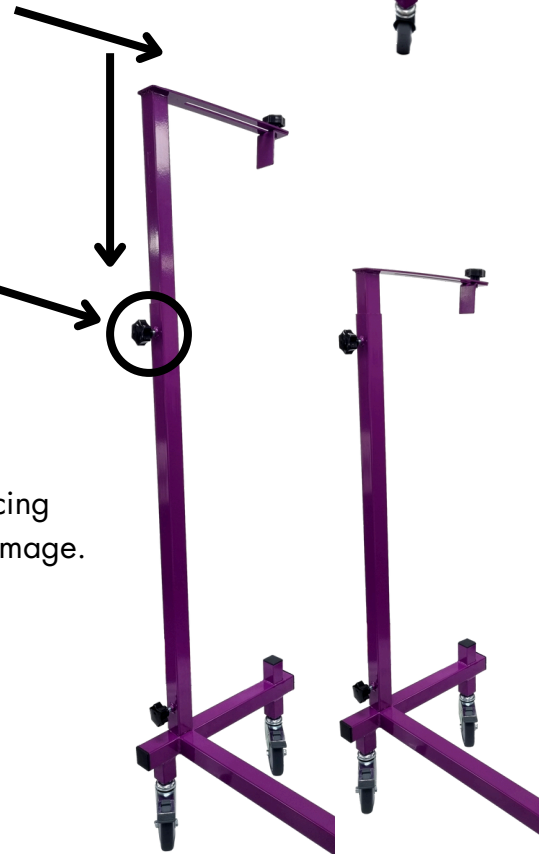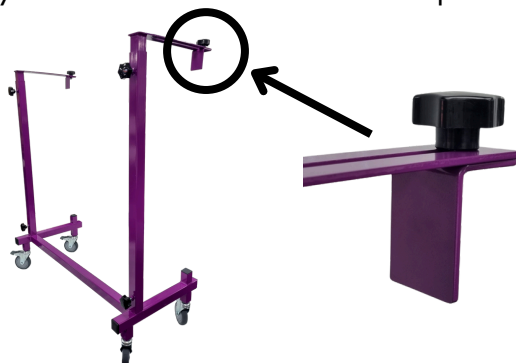

Standard usage.

Compatible with Icare Medical Group beds only, to maximum of Queen size.

180kg Max Load weight.

Do not use on rough surfaces or gravel.

Ensure wheels are facing the same way before loading to prevent from caster fatigue.